

INSTRUCTIONS #8 Dorsal Wrist Splint


Open straps. Straps may be repositioned on splint to facilitate closing by right or left hand. Position splint on top of hand and forearm with bend in dorsal bar aligned with wrist.

Comfortably secure straps. Evaluate fit, function and instruct patient on use.

Modifications– Remove splint. Bands and dorsal bar can be carefully bent to improve fit and function.

Clean with mild soap and damp towel. Air dry completely before reapplying.

Immediately contact your health care professional if you are experiencing loss of sensation, swelling, skin irritations or have any questions or concerns.

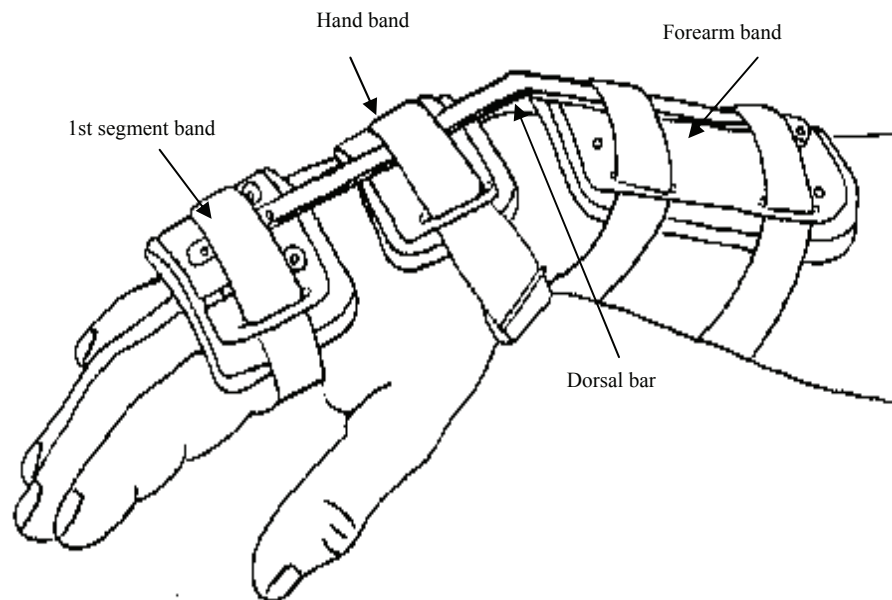

BUNNELL #8 Dorsal Wrist Splint to hold the wrist in flexion after repair of flexor tendons or after median and ulnar nerve suture.

CAUTION: Federal Law restricts this device to sale by or on the order of a physician



Single patient use